

DaysyView 2.0


Update and Synchronisation

1




Please update DaysyView in the appropriate store.

2

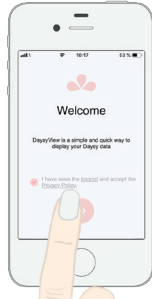

Confirm the privacy policy and click on the arrow to proceed to the next step.

3

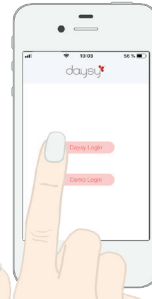


Select „Daysy-Login“. „Demo-Login“ is filled with sample data.

4




Connect Daysy to your smartphone, so that DaysyView can retrieve the serial number of your device.

5

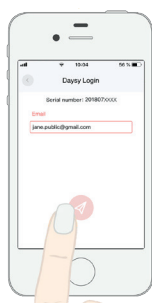


Enter and confirm your email address. A five-digit code will be sent to your stated email address.

6

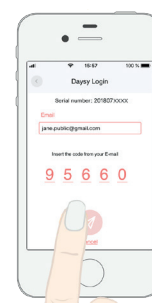

Enter the code from the email sent to you and confirm the details.

7

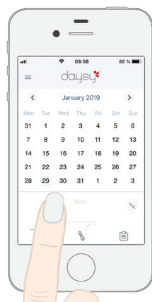

The update is complete when an empty calendar appears - you can now start synchronising.

8



Connect your phone to Daysy and wake her up. Synchronisation starts automatically. Please be patient. If Daysy falls asleep, activate her again.

9



If the current day is marked in color, you can disconnect the cable. Make sure Daysy is awake.